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Intended use
Light-curing lacquer for plastic dental applications

Indication for use
Light-curing lacquer for splints, partial and full dentures and temporary dental crowns in C&B technology.

Indication
For smooth coating of plastic applications, to prevent mechanical irritation of the tongue and mucosa.

Contraindication
An allergic reaction might occur in case of any allergy to (meth)acrylates.

Patient target group
Patients with loss of teeth and dental substance defects, as well as well as patients who require splint therapy with plastic structures.

Intended users
Dentists, dental technicians

Requirements / equipment
EyeVolution®, FotoDent® flash (with nitrogen)

Basic Material Properties
See technical datasheet (TDS). Technical datasheet available on request.

Specific Manufacturing Considerations
Caution
During processing, we recommend wearing personal protective equipment, e. g. protective work clothing, suitable gloves and safety goggles. Use a suitable extraction system or face mask during finishing.

Process description
Preparation
The area to be lacquered should be completely finished, dry and free from dust and grease residues. Due to the thin coat thickness, NanoVarnish is not suited as polish replacement. Therefore, the areas to be coated should be appropriately pre-polished.

Lot number / Expiry date
The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the

Further processing
For the best possible coating result, 10 MicroBrush are included in the package. These application aids ensure optimum distribution of even small amounts of material on the plastic surface. The lacquer should only be applied quickly and as a thin, uniform film. Perform intermediate polymerization, if required. Only take the required quantity out of the bottle, and immediately close the bottle.

Curing
EyeVolution® 7 min
FotoDent® flash (with nitrogen) 1 min pre-gas + 4 min

Disposal
Disposal of contents / container in accordance with local / regional / national / international regulations and according to the safety data sheet.

Hazards identification
Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Cleaning
The plastic applications should be cleaned before first use in the patient and for daily cleaning using specifically available cleaning agents.

Disinfection
We recommend the use of a 0.2 % chlorhexidine solution or 0.5 % sodium hypochlorite solution with an exposure time of 5 minutes for disinfection, and then allow to dry completely.

Sterilization
NanoVarnish cannot be sterilised.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

batch number. Do not use the product after expiration of the use-by date.

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