Maxima® by  HENRY SCHEIN®
900-6137

Non-alcoholic Surface disinfection

• Fast effective ready-to-use rapid surface disinfectant for disinfecting non-invasive medical device. Suitable for alcohol sensitive surfaces i.e. acrylic glass.

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CE 0473

750 ml e

*Non-alcoholic
Surface disinfection*

Indications for use: The spray comes with a dual spray and foam nozzle. Use on the foam setting. Spray the ready to use solution as foam on surfaces from a distance of 20-30 cm. With the use of a lint-free coloursafe cloth, thoroughly wipe over the surface and leave to dry. Excess foam to be removed with a disposable cloth. for use on larger surface areas, made of sensitive material such as ortified leather, test it first on a smaller area for compatibility. To avoid streaks from forming, a regular wiping with a moist (water) disposable cloth is recommended. **Microbiological Effectiveness/Effectivity:** Bacteria (incl. MRSA) in accordance to VAH/DGHM, EN 1040, EN 1276, EN 13697 (high bio load): Conc.: 1min. Candida albicans in accordance to VAH/DGHM, EN 1275, EN 1650, EN 13697 (high bio load): Conc.: 1min. membrane covered viruses (i.e. HBV, HIV, HCV, H1N1, H5N1) in accordance to BGA (RKI)/DW (with bio load): Conc.: 1min. (with bio load). Rota-viruses in accordance to BGVV/DW: Conc.: 1min. Noro-viruses in accordance to EN 14476 (clean): Conc.: 30min. **100g contains:** 0,24g Dialkyldimethylammoniumchlorid 0,48g Alkyl(Aryl)dimethylammoniumchlorid

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HENRY SCHEIN U.K. HOLDINGS LTD.