Material description

Laborsil clear is a highly transparent, addition-vulcanizing silicone with a final hardness of 70 Shore A for the universal use in the dental laboratory. Particularly suitable as a matrix material for light-curing anterior and posterior tooth composites.

Indication

For production of matrix

Processing

Laborsil clear is offered exclusively in our double cartridge system. By help of an injector and mixing tips an ideal mixing of the components without loss of time is guaranteed.

Application tip for Laborsil clear: In order to guarantee a preferably straight forwarding of the light through the form, the external surface should be smoothed. To use the optimal physical properties of Laborsil clear, a waiting time of approx. 10 min. should be maintained between curing and further processing. An insulation of Laborsil clear is not necessary.

Important processing instructions

For the correct use and in order to avoid problems during vulcanization of the material the following instructions should be followed:

- Dispense Laborsil material before putting the mixing canula on the double cartridge.
- Leave mixing canula on the cartridge for sealing effect.
- Polymerization of addition-vulcanizing silicones can be inhibited by latex gloves, creams, cleaning agents, resins etc.
- The product is exclusively designed for the field of application described above and only to be used by qualified and instructed personnel.

Until today, skin irritations or allergies are not known in connection with addition-vulcanizing silicone materials.

Keep away from children!

Only for use in the dental laboratory!

Storage conditions

At room temperature, i. e. 18 °C–28 °C

Batch number / use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

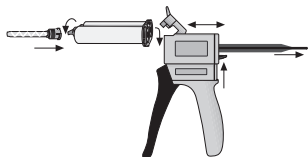
This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self-dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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