


DIRECTA ceramir®

Ceramir CAD/CAM BLOCKS Step by Step Guide


Preparation of the restoration



1
Remove the restoration
from the attachment
with a diamond bur

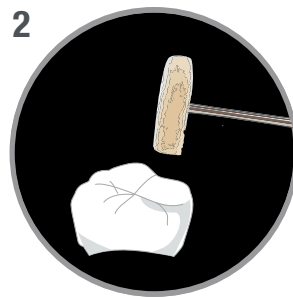


2
Final polish
with goat hair brush and
cotton / linen buffs

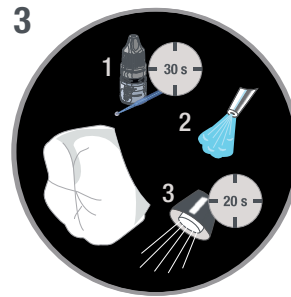


3
Prepare restoration
1. Apply a very thin layer of edelweiss VENEER
BOND to inner surface, rub surface for 30 sec
2. Dry surface with air
3. Light cure for 20 sec
(Or apply equivalent bonding/primer and use it
according to manufacturer's instructions.)

Preparation of the tooth

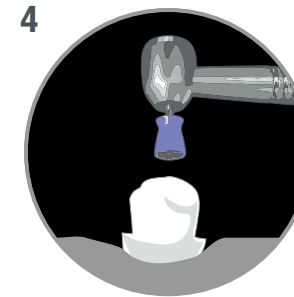


4
Clean the tooth
with polishing paste free from
oil and fluoride

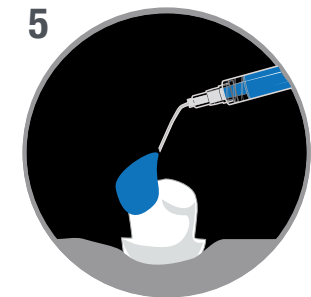


5
Apply
37% phosphoric acid
(Enamel 15-30 sec/
Dentin 15 sec)

Preparation of the tooth

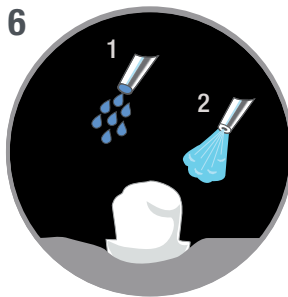


6
Rinse with water and
gently dry the surface

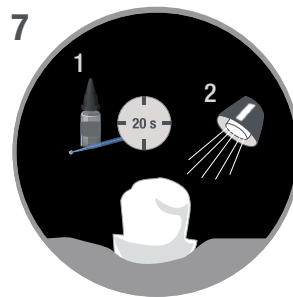

7
Prepare tooth
1. Apply dentin bonding (Rub surface
with dentin bonding for 20 sec.)
2. Light cure
(according to manufacturer's IFU)

Seating of the restoration

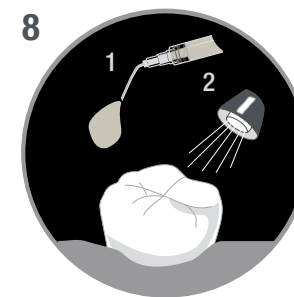

8
Cementation
1. Apply adhesive resin cement/
composite into restoration
and seat restoration
2. Light cure
(according to manufacturer's IFU)



9
Remove excess cement